

iSCREEN™ OFD COTININE

The iScreen™ OFD cotinine test offers fast and convenient oral fluid detection for cotinine. Oral fluid screening simplifies collection processes and protects against specimen tampering—allowing you to make informed decisions about smoking status.

FEATURES

- Detects up to 1-2 days after nicotine use
- Qualitative detection of cotinine in oral fluids at 30 ng/mL cutoff level
- Simple collection and testing procedure to determine recent nicotine use
- Long shelf-life at room temperature storage
- Results in as soon as 10 minutes
- Internal procedural control verifies invalid results




*For in vitro diagnostic use.
For the determination of nicotine use.*


TEST PROCEDURE AND INTERPRETATION

For detailed instructions, please read the package insert.


- 1** Remove cap and collect oral sample. Swab inside of the mouth with sponge for at least 3 minutes.

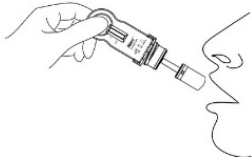

- 2** Replace cap on device. Twist handle clockwise 180° to tighten until red arrow lines up with other white mark.

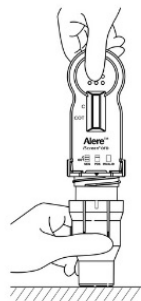

- 3** Place device on level surface and read at 10 minutes.

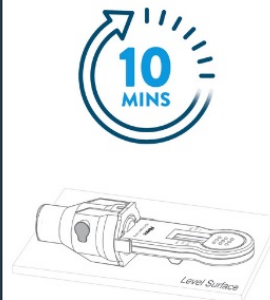

RESULTS INTERPRETATION

NEGATIVE



PRELIMINARY POSITIVE



INVALID

