# **BD Veritor™ System Reader**

## **Reader User's Manual**

L010108(03)

2013-09

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### **INTENDED USE**

The **BD Veritor™** System Reader is for use with **BD Veritor** System Test Kit Devices.

### **SUMMARY AND EXPLANATION**

The **BD Veritor** System Reader is a reflectometric reader that is part of the **BD Veritor** System. The **BD Veritor** System Reader detects the insertion of a **BD Veritor** System Test Kit Device and reads the device barcode to detect the assay type. Once it registers the assay type, the reader illuminates the device read window and measures the intensity of the control and analyte lines. Operator visual readings of the assay lack the sensitivity and objectivity of the reader, therefore assay result determination by operator visual readings of the **BD Veritor** System Test Kit Devices is not intended as part of the **BD Veritor** System.

### **PRINCIPLES OF THE PROCEDURE**

The **BD Veritor** System Reader is an optoelectronic instrument that uses a reflectance-based measurement method to evaluate the line signal intensities on the assay test strip, and a specific algorithm to determine the presence or absence of any target analyte(s).

The reader determines the line intensity at each of the spatially-defined test and control line positions, interprets the results using a scoring algorithm, and reports a positive, negative, or invalid result based on pre-set thresholds. A liquid crystal display

(LCD) on the instrument communicates status and results to the operator. The reader supports the use of different assays by reading an assay-specific barcode on the test device.

The instrument is maintenance free and is configured with a single button design (power switch).

## READER EXPIRATION

The reader is not configurable by the user and is designed to have a finite lifetime based on the number of tests performed, the number of days from first use by the end user, or a maximum shelf-life from the date of manufacture, whichever occurs first.

- Number of tests performed = 3000 tests maximum.
- Number of months **from first use** by the end user = a minimum of 22 months (670 days) up to a maximum of 24 months (730 days).
- Maximum shelf-life from the date of manufacture (regardless of use) = 34 months (1034 days).

## WARNINGS AND PRECAUTIONS

- For *in vitro* Diagnostic Use.
- **Do not use** after expiration date.
- Dispose of used **BD Veritor** System Test Kit Devices as biohazardous waste.
- Dispose of expired **BD Veritor** System Readers as biohazardous waste and according to local environmental electronic waste disposal regulations.
- Do not eat, drink, or smoke in the area where the specimens and kits are handled.
- Pathogenic microorganisms, including hepatitis viruses and Human Immunodeficiency Virus, may be present in clinical specimens. "Standard Precautions"<sup>1-4</sup> and institutional guidelines for the handling of biohazardous materials should be followed in handling all items contaminated with blood and other body fluids.
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are processed.
- Humidity and temperature can adversely affect results; therefore, operate the **BD Veritor** System Reader indoors at room temperature (15 – 30 °C).
- Protection provided by this equipment may be impaired if the equipment is used in a manner not consistent with the instructions in this manual.
- Replace the batteries in the **BD Veritor** System Reader with two new AA 1.5V non-rechargeable LR6 alkaline batteries. To reduce the risk of fire, do not use rechargeable batteries, lithium ion batteries, or other non-alkaline battery types in the **BD Veritor** System Reader.
- Note that the diagnostic port inside of the battery chamber is for BD use only. The plug on the diagnostics port should not be removed, nor should any cables be plugged into the **BD Veritor** System Reader.

- Ensure that the **BD Veritor** System Reader is not in direct sunlight or exposed to a bright light.
- The **BD Veritor** System Reader is designed for use only with **BD Veritor** System Test Kit Devices. Do not insert any foreign materials into the **BD Veritor** System Reader.

## PROCEDURES FOR SET-UP

### Box Contents

- 1 **BD Veritor** System Reader
- 1 User's Manual
- 1 **BD Veritor** System Verification Cartridge
- 2 AA 1.5V non-rechargeable LR6 alkaline batteries (installed)

**NOTE:** If any items are missing, contact your BD representative or BD Technical Services.

### Unpacking and Setup


1. Before using the **BD Veritor** System Reader, visually inspect the contents for damage. If damage is apparent, contact your BD representative or BD Technical Services.
2. Remove the reader from its protective wrapping. BD recommends retaining the packaging materials in a safe location.
3. Perform a verification test as described in the **BD Veritor** System Verification Cartridge section.

### Battery Installation

The reader requires two AA 1.5V non-rechargeable LR6 alkaline batteries such as DURACELL™ Coppertop MN1500 AA LR6, Eveready Energizer™ E91 Alkaline AA Cell LR6 AM3, or equivalent. To reduce the risk of fire, do not use rechargeable batteries, lithium ion batteries, or other non-alkaline battery types in the **BD Veritor** System Reader.

1. Using slight thumb pressure, slide the battery door, located on the bottom of the reader, toward the front as shown in Figure 1. Remove the door.
2. Remove the existing batteries and discard according to local environmental electronic waste disposal regulations.
3. Install the new batteries as shown in Figure 1. Ensure that the polarity orientation is correct as shown in Figure 1.
4. Reinstall the battery door.

Note that the diagnostic port inside the battery chamber is for BD use only. The plug on the diagnostics port should not be removed, nor should any cables be plugged into the **BD Veritor** System Reader.

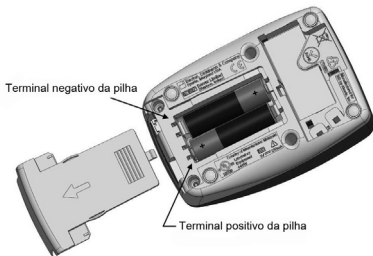**Figure 1 – Battery Access**

## **PROCEDURES FOR USE**

The **BD Veritor** System Reader is intended for use with the **BD Veritor** System Test Kit. Specific instructions and expiration dating (format: YYYY-MM-DD) can be found in the labeling included with each **BD Veritor** System Test Kit.

1. Place the **BD Veritor** System Reader on a flat, dry, stable surface.
2. Ensure that the **BD Veritor** System Reader is not in direct sunlight or exposed to a bright light.
3. Check the **BD Veritor** System Reader for an inserted device. If a device is present, remove it from the **BD Veritor** System Reader.
4. Press the front panel power button. The **BD Veritor** System Reader must complete a 5 second self-test before it is ready for use.
5. When the display window reads **INSERT TEST**, insert a **BD Veritor** System device into the slot on the right side of the reader as shown in Figure 2, aligning the insertion arrow on the device with the insertion arrow on the reader. Insert the device all the way in until it stops. A distinctive “click” will be felt when the device is properly engaged in the reader.
6. The display will show remaining read time as the read is performed. The read duration of the **BD Veritor** System Reader is approximately 10 seconds. Do not touch the **BD Veritor** System Reader or remove the **BD Veritor** System Test Kit Device during this time.
7. When the read is complete, the test result will be displayed with the name of the test and a “+” or “-” symbol to indicate positive test or negative test, respectively. Refer to the package insert for the **BD Veritor** System Kit for specific read interpretations.
8. Verify that the test name is correct.
9. Once the test name is verified and the result noted, remove the device by pulling it out. The display will change to **INSERT TEST** to indicate the reader is ready to perform another read.

10. Insert the next device and repeat steps 5 – 9 above, or turn off the power by pressing the front panel power button. If the reader is left unattended for 5 minutes, the power will turn off automatically and the test result will not be retained. After 5 minutes, the **BD Veritor** System Reader will automatically shut off and steps 2 – 9 must be repeated.

**Figure 2 – Device insertion and removal**

## **PROCEDURES FOR MAINTENANCE AND SERVICING**

### **BD Veritor System Verification Cartridge**

A Verification Cartridge is supplied to allow the user to perform a functional test on the reader. The frequency of verification testing is dictated by local site regulations or practices. A verification test does count as one test for the reader's maximum allowed number of tests. All **BD Veritor** System Verification Cartridges must be obtained from BD or from a BD-authorized distributor. Cartridges from other manufacturers are not compatible with the **BD Veritor** System Reader.

To perform a verification test:

1. Place the **BD Veritor** System Reader on a flat, dry, stable surface.
2. Ensure that the **BD Veritor** System Reader is not in direct sunlight or exposed to a bright light.
3. Check the **BD Veritor** System Reader for an inserted device. If a device is present, remove it from the **BD Veritor** System Reader.
4. Press the front panel power button. The **BD Veritor** System Reader must complete a 5 second self-test before it is ready for use.
5. When the display window reads **INSERT TEST**, remove the **BD Veritor** System Verification Cartridge from the pouch and insert into the slot on the right side of the reader as shown in Figure 2, aligning the insertion arrow on the cartridge with the insertion arrow on the reader. Insert the cartridge all the way in until it stops. A distinctive "click" will be felt when the cartridge is properly engaged in the reader.

6. The display will show remaining read time as the read is performed. The read duration of the **BD Veritor** System Reader is approximately 10 seconds. Do not remove or touch the cartridge during this time.
7. When the read is complete, the test result will be displayed as one of the two following messages:  
**QC PASS**  
**QC FAIL**
8. If the message **QC PASS** appears, the reader may be used to test specimens. If the message **QC FAIL** appears, the reader may not be used to test specimens and must be replaced. Please contact BD Technical Services at (800) 638-8663 for assistance.
9. Return the **BD Veritor** System Verification Cartridge to the resealable pouch for the next use.

### **Cleaning**

The outer case and display may be wiped with a clean towel lightly moistened with 70% isopropyl alcohol (IPA), 10% bleach solution, or mild soap solution. Do not introduce cleaning solution or any other liquid directly into the unit. Do not use a saturated towel which may introduce liquid into the case or display seams. Ensure that the **BD Veritor** System Reader is dry and the surface is free of fluid prior to returning to use.

### **Servicing**

There are no user serviceable components in the unit with the exception of the replaceable batteries. For technical issues or questions, please contact BD.

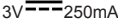
### **Transportation and Storage**

The **BD Veritor** System kit can be stored at room temperature or refrigerated (2 – 30 °C). **DO NOT USE** beyond the expiration date.

The **BD Veritor** System Reader should be stored at 15 – 30 °C.

## SYMBOL DEFINITION

Symbol	Description
	Reader power actuation (ON/OFF)
	Caution, risk of danger (Consult reader manual for warnings and precautions)
	Underwriters Laboratory (UL) listing mark
	Reader serial number (13 digits)
	BD catalog number (6 digits)
	Material recycling classification
	This device should be treated as waste equipment and disposed of at a designated collection point
	Use by (YYYY-MM)
	Authorized Representative in the European Community
	Temperature Limitation

Symbol	Description
	Compliance with the Restriction of Hazardous Substances (RoHS) Directive
	Reader rated at 3V and 250mA
	Biological hazard
	Reader manufacturer of record
	Reader model number (AXX-X)
	Date of manufacture (YYYY-MM)
	Batch Code (Lot)
	In Vitro Diagnostic Medical Device
	Consult Instructions for Use

## MESSAGES

Messages displayed by the **BD Veritor** System Reader are described in the table below. For assay-specific messages, see the appropriate package insert.

Message	Type	Meaning	Action Needed
ANALYSIS WAIT XXs	Status	Read in progress, where XX is the total time remaining	None
BARCODE INVALID	Error	The device barcode cannot be read correctly	Repeat the assay. On subsequent occurrences, contact BD.
RESULT INVALID	Error	Result invalid	Repeat assay
CONTROL INVALID	Error	One or more of the assay control lines was outside of the normal range	On first occurrence, repeat the assay. On subsequent occurrences, contact BD.
ERROR 5 BASELINE	Error	The assay background area was outside of the normal range	Remove the device and cycle the instrument power. <ul style="list-style-type: none"><li>• If the instrument displays an error during power-on, contact BD.</li><li>• If no error message appears, obtain an additional sample and repeat the assay. If the error recurs, contact BD.</li></ul>
INSERT TEST	Status	Ready to read	Insert a device
NO TESTS REMAIN	Error	The maximum number of reader tests has been performed	Contact your distributor to order a new reader
READING WAIT XXs	Status	Read in progress, where XX is the total time remaining	None
REMOVE TEST	Status	A device is inserted in the reader	Remove device
QC FAIL	Error	Result of a verification test	Contact BD
QC PASS	Status	Result of a verification test	None
READER EXPIRED	Error	The operational life of the reader is expired	Contact your distributor to order a new reader

Message	Type	Meaning	Action Needed
READER FAILURE	Error	A hardware failure occurred	Contact BD
REPLACE BATTERY	Error	The batteries are depleted	Replace the batteries
SELFTEST vX.XX	Status	Self-test in progress. "X.XX" is the reader version	None
START-UP WAIT XXs	Status	Read in progress, where XX is the total time remaining	None
TEST WAS REMOVED	Error	The device was removed while the read was in progress	That read is invalidated. The device may be re-inserted to begin a new read.
XX DAYS REMAIN	Warning	"XX" is the number of days until reader expiration	Contact your distributor to order a new reader
XX TESTS REMAIN	Warning	"XX" is the number of reads before reader expiration	Contact your distributor to order a new reader

## SPECIFICATIONS

<b>Battery Ratings:</b>	Two AA 1.5V non-rechargeable LR6 alkaline batteries
<b>Ambient Temperature:</b>	15 – 30 °C
<b>Ambient Humidity:</b>	10 – 85% (relative humidity, non-condensing)
<b>Ambient Light:</b>	0 – 1000 lux
<b>Ambient Environment:</b>	Indoor Use Only
<b>Installation Category:</b>	II (per IEC 664)
<b>Pollution Degree:</b>	2 (per IEC 664)
<b>Altitude:</b>	0 – 3,000 m
<b>Length:</b>	12 cm
<b>Width:</b>	8 cm
<b>Height:</b>	6 cm
<b>Weight:</b>	0.5 kg
<b>Lifetime:</b>	The earliest of: <ul style="list-style-type: none"> <li>• 3,000 reads</li> <li>• minimum of 22 months (670 days) up to a maximum of 24 months (730 days) from first use</li> <li>• 34 months (1034 days) from production (regardless of use)</li> </ul>

## AVAILABILITY

Cat. No.	Description
----------	-------------

256055	<b>BD Veritor™</b> System Reader
--------	----------------------------------

256045	<b>BD Veritor™</b> System for Rapid Detection of Flu A+B, 30 tests (CLIA-waived kit)
--------	--------------------------------------------------------------------------------------

256041	<b>BD Veritor™</b> System for Rapid Detection of Flu A+B, 30 tests
--------	--------------------------------------------------------------------

256042	<b>BD Veritor™</b> System for Rapid Detection of RSV
--------	------------------------------------------------------

For ordering information regarding spare parts (including the **BD Veritor™** System Verification Cartridge), please contact BD Technical Services at 800-638-8663, or your local BD representative.

## REFERENCES

1. Clinical and Laboratory Standards Institute. 2005. Approved Guideline M29-A3. Protection of laboratory workers from occupationally acquired infections, 3rd ed. CLSI, Wayne, Pa.
2. Garner, J.S. 1996. Hospital Infection Control Practices Advisory Committee, U.S. Department of Health and Human Services, Centers for Disease Control and Prevention. Guideline for isolation precautions in hospitals. *Infect. Control Hospital Epidemiol.* 17:53-80.
3. U.S. Department of Health and Human Services. 2007. Biosafety in microbiological and biomedical laboratories, HHS Publication (CDC), 5th ed. U.S. Government Printing Office, Washington, D.C.
4. Directive 2000/54/EC of the European Parliament and of the Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work (seventh individual directive within the meaning of Article 16(1) of Directive 89/391/EEC). *Official Journal L262, 17/10/2000, p. 0021-0045.*

Technical Information: In the United States, contact BD Technical Service and Support at 800-638-8663 or [www.bd.com/ds](http://www.bd.com/ds).

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© 2013 BD.
DURACELL is a registered trademark of The Gillette Company. Energizer is a registered trademark of Eveready Battery Company.
BD, BD Logo and BD Veritor are trademarks of Becton, Dickinson and Company.
Benex Limited