

iSCREEN™ ORAL FLUID TEST DRUG SCREEN CUBE

The iScreen™ Oral Fluid Test Drug Screen Cube is intended for employment and insurance testing, and offers fast and convenient oral fluid detection for multiple drug assays. Oral fluid drug screening simplifies collection processes and protects against specimen tampering—allowing you to make informed decisions about substance abuse.

FEATURES

- Detects 5-10 drugs in oral fluid – multiple configurations available allowing for a fully customized oral fluid drug testing program
- Ensures sufficient specimen volume collection with an easy to read saliva saturation indicator
- Eliminates the need for gender-specific observations and limits adulteration and tampering
- Provides easy to read results in 5 minutes that can be photocopied for result recording

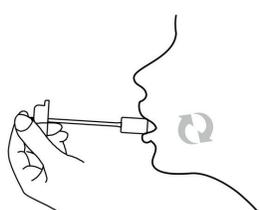


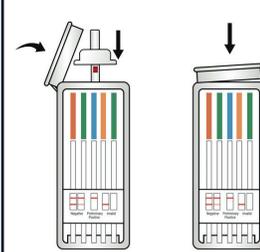
*For in vitro diagnostic use.
For use in employment and insurance testing.*

TEST PROCEDURE AND INTERPRETATION

For detailed instructions, please read the package insert.

- Put collector sponge on tongue or near cheek to collect oral fluid for about 3 minutes (until saturation indicator strip appears red)


- Open the test cube and place the saturated collector inside. Press firmly until it reaches the bottom. Close the test tube lid. Keep cube upright on a flat surface


- Compare adulteration results to color chart Interpret drug test results at 5 minutes.



RESULTS INTERPRETATION

NEGATIVE



PRELIMINARY POSITIVE



INVALID



* Some drug analytes and cutoffs may not be available in all locations, please consult the product insert or your sales representative for more details.