

Cautions

- Rx Only: Federal (USA) law prohibits this device for sale by or on the order of a physician. Not for Drug Storage.
- Does not contain natural latex.
- Do not use if the packaging is opened or damaged.
- Do not use if the Pre-Filled Normal Saline Flush Syringe is damaged or leaking.
- Do not use if the nozzle cap is not installed correctly or separated.
- Do not use if the solution is discoloured, turbid, precipitated or contains any form of suspended particles by visual inspection.
- Do not re-sterilize.
- Do not use if the device is beyond the expiry date.
- Do not use the device in a sterile interface or a sterile environment.
- For single use only, do not reuse. Discard all unused remaining parts.
- Do no contact the solution with incompatible medicines.

Storage

- Device should be stored between the temperature of 59-86°F / 15-30°C.

Graphs and Symbols used for Medical Device Labels

	Single use		Temperature limit
	Don't use if the package is damaged.		Keep dry
	Consult instructions for use		Date of manufacture
	Manufacturer		Use-by date
	Lot number		Do not re-sterilize
	Non-pyrogenic		Unique Device Identifier
	Sterilized using steam or dry heat		Keep away from sunlight
	This way up		Stacking limit by mass