



Instructions for Use - Pre-Filled Normal Saline Flush Syringe

Indications for Use

- The Pre-Filled Normal Saline Flush Syringe is intended to be used for the flushing of the central and peripheral vascular access devices.

Product Description

- The Pre-Filled Normal Saline Flush Syringe is a three-piece, sterile, single use syringe with a 6% conical taper luer connector, prefilled with 0.9% sodium chloride injection, and sealed with a nozzle cap.
- The Pre-Filled Normal Saline Flush Syringe is provided with a sterile fluid path, sterilized via moist heat.

Product Components

- Components:- barrel, plunger, piston, nozzle cap and 0.9% sodium chloride injection (conforming with USP standards).

Product Specification

- 10ml.

Sterilization Method

- Moist heat sterilization.

Shelf Life

- 3 years.

Instructions for Use

- Step 1: Remove the Pre-Filled Normal Saline Flush Syringe from the packaging.
- Step 2: Hold the Pre-Filled Normal Saline Flush Syringe upwardly, unscrew the nozzle cap and expel the air.
- Step 3: If the product is connected to an intravenous infusion system with a needleless infusion connector, please connect the needleless infusion connector directly. If the product is connected to a traditional infusion connector, requiring a steel needle connection, connect a safety needle that meets the requirements of the OSHA bloodborne pathogen standard.
- Step 4: Connect the product to the connector valve, or needleless system, and flush relevant to the catheter design.
- Step 5: Used Pre-Filled Normal Saline Flush Syringe should be disposed into an approved sharps container.

Contraindications

- N/A