

iSCREEN™ AND iSCREEN™ Dx URINE TEST DRUG SCREEN SQUARE CUP

The iScreen™ and iScreen™ Dx Urine Test Drug Screen Square Cup is a convenient all-inclusive, onsite urine drug screening device that simplifies the collections process and tests for up to 14 drugs. The self-contained cup minimizes collector exposure to urine, while the device provides fast and reliable results within five minutes that can be photocopied for result recording. Specimen Validity/ Adulteration test strips are also available.

FEATURES

- Integrated test cup screens for 4-14 drugs: AMP, BAR, BUP, BZO, COC, MDMA, MET, MOP, MTD, OPI, OXY, PCP, PPX, TCA, THC
- Specimen validity combinations include: CR, NI, PH, OX, SG
- Self-contained collection cup and test help minimize exposure to urine
- Fast results in 5 minutes



*For professional, in vitro diagnostic use.
CLIA waived options available.*

TEST PROCEDURE AND INTERPRETATION

For detailed instructions, please read the package insert.

- 1 Collect sample and attach lid


- 2 Wait 5 minutes to read results.



Drug and adulterant strip placement on the cup may vary depending on configuration.
- 3 Compare adulteration results to color chart. Interpret drug test results at 5 minutes



RESULTS INTERPRETATION

NEGATIVE



PRELIMINARY POSITIVE



INVALID



* Some drug analytes and cutoffs may not be available in all locations, please consult the product insert or your sales representative for more details.